Mini BIO

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Mini BIO

Auto-Reader

For incubation and readout of Rapid or Super Rapid Biological Indicators

Usage

For incubation and readout of:

Bionova® Rapid Fluorescence Readout Biological Indicators:

BT220 (Steam): 3 hrs. at 60 °C, BT221 (Steam): 1 hr. at 60 °C and

BT110 (EO): 4 hrs. at 37 °C. BT102 (Formaldehyde): 2 hrs at 60°C.

BT95 (VH2O2): 2 hrs at 60°C.

Bionova® Super Rapid Fluorescence Readout Biological Indicators:

BT222 (Steam): 1 hr at 60 °C.

BT223 (Steam): 30 min at 60 °C.

Applicable Regulation

European Union Low Voltage Directive 2014/35/EU.

Electromagnetic Compatibility Directive 2014/30/CE.

RoHS Directive 2011/65/EU.

Authorization

Designed under Quality Management System standards ISO 13485:2003/NS-EN ISO 13485:2012.

Characteristics

3 metal positions (38 mm deep, 9 mm diameter) to incubate Rapid/Super Rapid Fluorescence Readout Self-contained biological indicators.

1 ampoule crusher.

Special hole for external thermometer.

Incorporated top cap.

Dual Temperature System: Allows selecting between two different incubation temperatures (37°C and 60°C)

USB connection for registering results in a PC through Reading and Traceability Software.

Allows incubating 3 indicators simultaneously, in different incubation times, at the same temperature.

Thermal Printer to record results.

Sound alarm for event indication

Detection and automatic cancellation of the fluorescence reading of the biological indicator.

Fluorescence reading wavelength: ~ 460 nm

Firmware update by the user.

Dimensions: 180 mm high, 154 mm wide and 105 mm deep.

Voltage range: 100 – 240 V

Power: 28 W

Frequency: 50-60 Hz

Environmental conditions during manufacture

Room temperature 15-35 °C, RH 35-85 %

Storage conditions

T = 10-30 °C, RH 30-80 %, keep in a dark place.

Transportation conditions

Storage conditions should be strictly followed. The product should be transported in closed and reinforced boxes in order to avoid damages. Product transportation does not represent any risk for human health.

Shelf-life

10 years

Packing

One unit per box.

Packing information: product code and description, and manufacturer information.

Labelling

Same label on product and product's box: product code and description, batch number, serial number and manufacturer information.

Possible target markets

Healthcare, Food, Pharmaceutical and medical devices industries.

Other important information (optional)

Bionova® Mini BIO has a button located on the incubator panel to select temperatures at 37 ° C or 60 ° C. It also has a button for each of the positions of readout to set up a different time for each position. Bionova® Mini BIO incubator has an automatic readout system and a printer to record the results.

Note: When necessary, limits of each product's parameters should be specified.

Description of identified and/or predictable risks which could represent a risk for the product

Do not pour any liquid inside.

Do not immerse into any liquid.

Only for indoor use.

Disconnect the power cord before cleaning.

Do not use abrasive or corrosive cleaners or disinfectants.

Make sure that the incubator is connected to a properly rated power cord.